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IMOPG/1U/09/2019

Carefully read the instructions for use contained in the package, as it contains important information for the patient about the use of the product Onco Immufen gel on skin after radiotherapy, and then store it in case the need to re-read.

During chemo and radiotherapy act in accordance with doctor's instructions.

Instructions for use Onco Immufen gel on skin after radiotherapy Medical device 50 ml

1. What is Onco Immufen gel on skin after radiotherapy

Onco Immufen gel on skin after radiotherapy is a medical device in form of gel used for proper moisturizing, lubricating and creating of protective barrier near the places subjected to radiotherapy. Due to creation of proper environment it supports course of natural regenerating processes.

2. What is Onco Immufen gel on skin after radiotherapy used for

One of many side effects of radiotherapy are skin changes. They are caused by distortion of skin barrier function. These changes include: rash, skin dryness, hyperpigmentation, inflammations, desquamation of the skin etc. Moreover skin is more prone to infections and allergy and more sensitive to UV radiation. Skin problems can cause: pain, strong discomfort, diminish quality of life and can be reason of therapy termination or its delay. Therefore very important is proper skin care, primarily its moisturizing and lubricating. Proper skin hydration reduces dryness, tension and itching, thereby protects from infection caused by skin scratching.

Onco Immufen gel on skin after radiotherapy contains numerous

moisturizing substances like glycerin, arginine and sodium hyaluronate and strongly lubricating oil Omega Plus complex. Additionally contained in the device Syrcalm™ protects from excessive water loss and strengthens skin protective barrier. Thanks to this the skin is more elastic, moistened and less prone to action of external factors, what helps to reduce feeling of tension and itching. Due to proper skin hydration, the appropriate environment is maintained, which favors natural regenerating processes.

Moreover contained in the device - prebiotic Biolin P - ensures adequate environment for physiological bacterial flora development, what protects from development of pathogenic microorganisms. This prevents oncological patient sensitive skin from infection.

3. Dosage and how to use

Onco Immufen gel on skin after radiotherapy should be used according to instructions for use.

The state and condition of the skin should be observed both, during and after basic therapy. In case of appearance of any worrying symptoms or changes on the skin, contact your doctor immediately.

Caution! Do not use directly before radiotherapy as it can cause irritation of the skin. The interval between using of gel and radiotherapy should be at least few hours.

Do not use on open not-healed wounds and for the injury with infraction of dermis.

In case of any questions contact your physician.

How to use:

Small amount of gel apply on the clean skin, which is subjected to radiotherapy. Spread gently, maintaining hygiene rules. Use as often as skin requires, up to few times a day, especially few hour after radiotherapy and during the night.

4. Contraindications

Hypersensitivity (allergy) to product ingredients.

5. Storage conditions

Store in original container in room temperature below 25°C.

Keep out of reach and sight of children.

6. Composition

Main ingredients: glycerin, PEG-40 hydrogenated castor oil, Syrcalm™ (Phragmites kharka extract, Poria cocos extract), Biolin P (inulin, alpha-glucan oligosaccharide), sodium hyaluronate, Omega Plus Complex (Helianthus annuus seed oil, Zea mays oil, Sesamum indicum seed oil, Macadamia integrifolia seed oil, Olea europaea fruit oil), arginine.

Additive ingredients: aqua, carbomer, hydroxyacetophenone, 1,2-hexanediol, caprylyl glycol, tocopheryl Acetate, sodium benzoate.

7. Packaging description

Onco Immufen gel on skin after radiotherapy is packed in bottle with 50 ml of device.

The unit packaging contains: box, bottle airless and instructions for use.

Manufacturer:

Gofarm Sp. z o.o. Sp.k., Górników 21/26 street, 30-819 Kraków
www.gofarm.pl

Manufactured for:

Sanagen Pharm, Borongajska cesta 83 H, Zagreb, Croatia
www.immufen.com

Symbols presented on the packaging

Manufacturer

Expiration date

Batch number

See instructions for use

Maximal storage temperature

Medical device

Do not use if the package is damaged

Date of instructions for use preparation: September 2019
IMOPG/1U/09/2019

Pažljivo pročitajte priloženo uputstvo za upotrebu medicinskog sredstva Onco Immufen gel on skin after radiotherapy jer sadrži važne informacije za pacijente o njegovoj upotrebi. Uputstvo sačuvajte za slučaj da treba ponovo da ga pročitate.

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