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Datum revizije uputstva: August 2019.

IMHFSC/IU/08/2019

Carefully read the instructions for use contained in the package, as it contains important information for the patient about the use of the product **Immufen - Onco Hand-Foot Syndrome Cream**, and then store it in case the need to re-read.

During chemo and radiotherapy act in accordance with doctor's instructions.

Instructions for use
Immufen - Onco Hand-Foot Syndrome Cream
Medical device
100 ml

1. What is Immufen - Onco Hand-Foot Syndrome Cream

Immufen - Onco Hand-Foot Syndrome Cream is a medical device in form of cream, rich in ingredients which form on skin moisturising-lubricating protective barrier. Created film, eases course of side effect, of some chemotherapeutics and biological medicines, like Hand-Foot Syndrome (palmar plantar erythrodysesthesia). The cream should be used as support for main oncological therapy (chemotherapy).

2. What is Immufen - Onco Hand-Foot Syndrome Cream used for ?

Hand-Foot Syndrome is a side effect of chemotherapy, characteristic for many currently used chemotherapeutics and biological medicines. It starts with tingling sensation, burning and stiffening on the hands and feet surface. Afterwards redness and proceeding pain appears. Other symptoms of that syndrome are: keratinization, desquamation, protrusion and fracture of the skin. Also blisters and ulcers can appear. These symptoms not only influence the life comfort of the patients but also when are very serious can cause lowering of therapeutic dose or even temporary treatment termination. Therefore very important is proper protection of hand and feet to prevent and ease of occurring symptoms of syndrome. It is recommended to avoid strong skin irritating substances like perfumes, forceful washing compounds, hot baths, wearing too tight shoes and excessive sun exposure. It is also recommended to regular use of moisturizing and softening creams, which prevent keratinization of the skin, particular those containing urea.

Immufen - Onco Hand-Foot Syndrome Cream contains 10% of urea, which intensely moisturises and softens irritated skin. Additionally contained in the product substances like Omega Plus Complex, Shea butter, silk proteins or SyriCalm, lubricate and moisturise skin, so that ensuring protective barrier from harmful external factors. They also create proper environment for natural regenerating processes. Proper moisturizing and lubrication of the skin, decreasing symptoms of itching, burning and tightness.

3. Dosage and how to use

Immufen - Onco Hand-Foot Syndrome Cream should be used according to instructions for use.

How to use:

Use on clean and undamaged skin. Small amount of cream apply to the skin of your hands and feet. Gently spread. Start the application few days before first chemotherapy or biological medicines intake. Then use regularly, few times a day according to needs, especially after washing and drying of hands and feet and after using irritating agents. After termination of basic therapy the cream should be used for at least 2 weeks.

In case of any questions contact your physician.

The device is intended for adults and children over 3 years of age.

4. Contraindications

Hypersensitivity (allergy) to product ingredients.
Do not use on open not-healed wounds and for the injury with infraction of dermis.
Avoid contact with eyes.

5. Storage conditions

Store in original container in room temperature below 25°C.
Keep out of reach and sight of children.

6. Composition

Main ingredients: urea, SyriCalm™ CLR (extracts from Phragmites kharka and Poria cocos), shea butter, oil complex Omega Plus (Helianthus annuus seed oil, Zea mays oil, Sesamum indicum seed oil, Macadamia integrifolia seed oil, Olea europaea fruit oil), hydrolyzed silk, glycine soja oil.

Additive ingredients: aqua, glycerin, petrolatum, cetearyl alcohol, glycerol stearate, phenoxyethanol, ethylhexylglycerin, stearyl sodium glutamate, sodium polyacrylate, aroma of white tea, disodium EDTA, beeswax, triethyl citrate, stabilizer (PEG-8, tocopherol, ascorbyl palmitate, ascorbic acid, citric acid), imidazolidinyl urea, potassium sorbate, sodium benzoate.

7. Packaging description

Immufen - Onco Hand-Foot Syndrome Cream is packed in plastic tube with 100 ml of device.

The unit packaging contains: box, tube and instructions for use.

Manufacturer:

Gofarm Sp. z o.o. Sp.k.
Górników 21/26 street
30-819 Kraków
www.gofarm.pl

Manufactured for:

Sanagen Pharm Ltd, Borongajska cesta 83 H, Zagreb, Croatia
www.immufen.com

Symbols presented on the packaging

Manufacturer

Expiration date

Batch number

See instructions for use

Maximal storage temperature

Medical device

Do not use if the package is damaged

Date of instructions for use actualization: August 2019
IMHFSC/IU/08/2019